

MALFARBEN

STAINS



WOHLWEND AG

Arbeitsanleitung

Bevor die Malfarben aufgetragen werden, muss die Porzellanarbeit sorgfältig gereinigt und getrocknet werden. Die Malfarben werden mit der entsprechenden Flüssigkeit vorgemischt.

Mit dem Malfarbenpinsel wird zunächst die Oberfläche mit Flüssigkeit angefeuchtet. Dadurch erscheinen die Farben wie nach dem Glanzbrand. Führen Sie alle Farbangleichungen oder Charakterisierungen in gewünschtem Umfang durch. Setzen Sie nun die Arbeit auf einen Brennträger vor den geöffneten Brennofen zum Trocknen vor dem Brand.

Instruction for use

Before applying surface stains the restoration must be thoroughly cleaned and dried. Premix the stains with staining liquid.


With the included stain brush, add staining medium liquid to the surface of the crowns. This will permit you to see the colour as it would appear after glazing. Make any colour adjustments or characterizations required then put on firing tray and place at furnace entrance for drying and firing.

MALFARBEN

STAINS

Wohlwend AG
Dental Manufaktur

Platta 52
FL-9488 Schellenberg
Fürstentum Liechtenstein
Phone: +423-373-4243
Fax: +423-373-4244
www.wohlwend-ag.com
info@wohlwend-ag.com

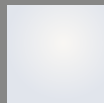

glasur
glaze



blau
blue



erdgrau
grey



oliv
olive



honig
honey



kastanie
chestnut



rotbraun
redbrown



weiß
white