LUKA**Steam S**

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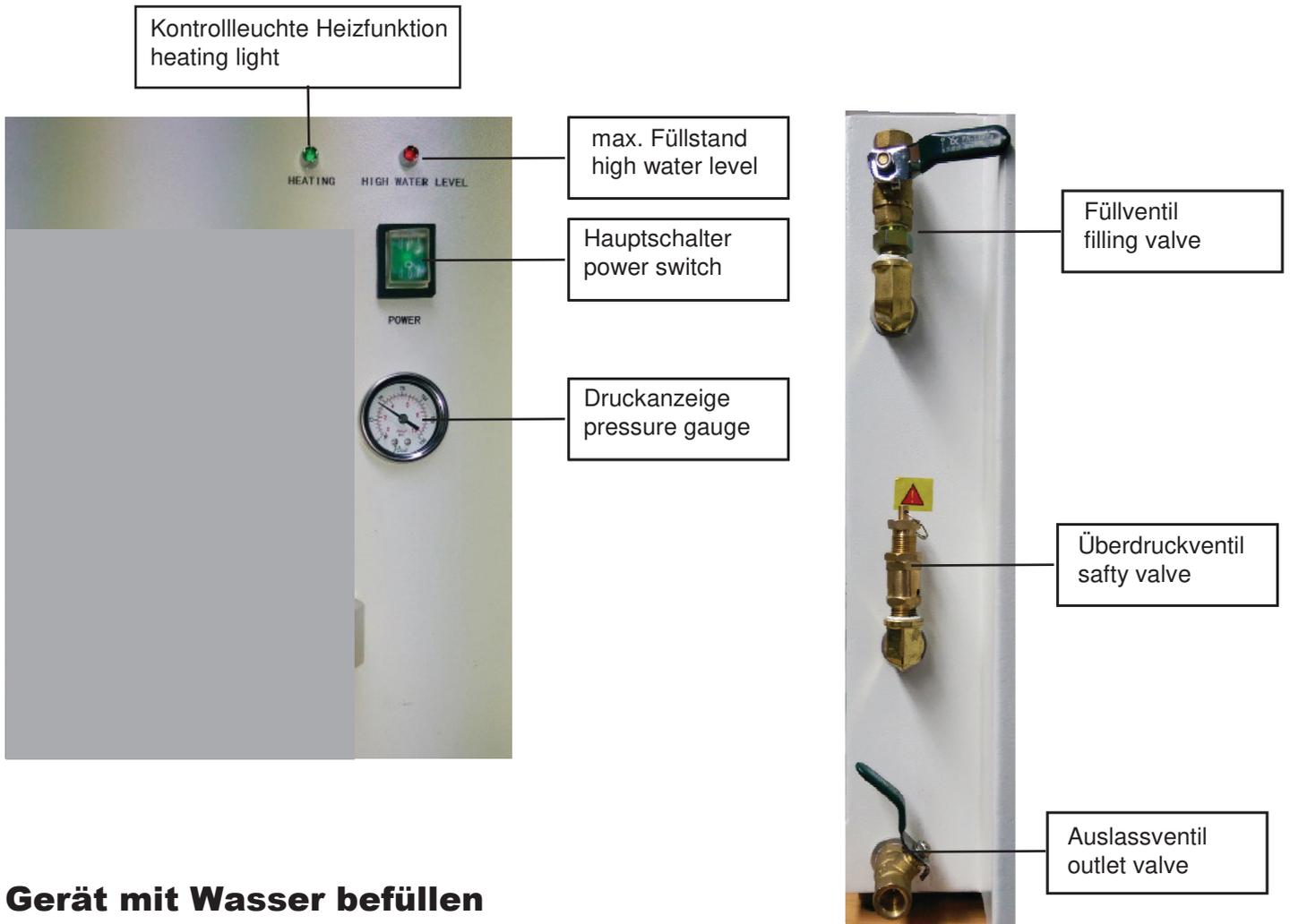
Before connecting the steam cleaner and apply,
please read these instructions!

Lukadent GmbH
Felsenbergweg 2
D-71701 Schieberdingen
+49 (0) 7150-32955
info@Lukadent.de

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Green funnel to fill valve screws. Turn on a device. When completely empty tank beeps an alarm sound. Press footswitch and pour water. Once enough water in the tank, the heating process begins automatically. As long as water fill up the red light „high water level“ indicator lights. The tank is now filled with about 6 liters of water.

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Turn on the appliance at the main switch. Sounds an audible alarm, please fill it with water (see above). Otherwise, the heating process will start automatically. We recommend the unit in the morning to refill when the operation starts. The heating process takes about 10 minutes, then the device is ready for operation. The large 6-liter water tank can be a very long time working with the unit. Triggering the steam-working process is carried out by means of the foot switch. Please note the safety instructions.

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The Lukas Team S is a steam-cleaning appliance and intended for commercial use in the dental laboratory!

This device is built according to the current state of technology and when used properly and safe to operate. However, going out, when this unit is improperly installed or used for an improper use of this device hazards!

The control and operating personnel is basically familiar with the steam-cleaning process. The control and operating personnel is basically familiar with the possible hazard potential in dealing with pressurized steam-cleaning equipment. Should the listed here know today, a provided operator is not available, so the equipment operator for the provision of the required qualifications of its employees is responsible!

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This unit is equipped to produce steam with a closed pressure vessel, this results in a permissible operating pressure of about 5 bar and an operating temperature of about 160 °C.

The device is equipped with a temperature limiter (175 °C) and with a safety valve (opens at 7 bar). Even when a malfunction in the device, for example, when a malfunction of the heating control or at too high a water filling, no risk to the equipment and the operating personnel may occur. The equipment operator is responsible that the integrated safety function remain functional ready the device over the entire operating period. Check at least once a month, the function of the safety valve! Let perform an equipment inspection with safety testing of the device and the pressure vessel innsbesondere every 2 years.

Volt	Herz	Watt	Druck /pressure max	Kg	Size
230	50	1600	5 bar	15	42x30x37