LUKAVEST Hardener

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Cold-dipping hardener liquid.

Uses

For investment duplicated models in Cr-Co technique.

Advantages

Easy to use, supplied ready to use and does not need to be diluted. No dilution required.

- Dries within a few minutes.
- It is a clean and economical alternative to the conventional waxing process.
- LUKAVest Hardner penetrates the surface of the model, acts as a binder and leaves the entire surface hardened, preventing any coating residue from coming off.

Working instructions

Fill a plastic or metallic receptacle (or one made by any alcohol resistant material) with an amount of product that covers a minimum high of 10 cm.

First of all, dry the duplicated investment models at 220-240 °C for 30-40 minutes following the instructions of the investment manufacturer

Once finished the drying process, dip the hot model in the receptacle with LUKVest Hardner and maintain for 3 seconds. If you want a thicker and harder coating repeat for 3 more seconds.

If an adherent surface is needed, to make easier therefore the later adaptation of the wax on the model, you can paint with a brush (1 or 2 layer) when the model had get cold.

Let dry the model outdoors until it cools off.

Cautions

1.- USE CAUTIONS:

LUKAVest Hardner is prepared to be used cold, NEVER hot. LUKAVest Hardneris a flammable product and it must be used with the habitual precautions of this type of products (see the security sheet). Choose a place away from heat sources and naked flames (furnaces or burners)

After each use, cover the container to avoid the evaporation of the dissolvent. An excessive evaporation of the dissolvent can cause the sedimentation of the incorporated resin causing models with excessive thickness. In such case, replace the product by a new one.

2.- SAFETY CAUTIONS:

Irritant and flammable product.

Keep away from children.

Handle the product in a well-ventilated place.

The handling of this product has to be done only by qualified people In the event of ingestion, consult a doctor.

For further information, see the safety data sheet for the product.

Storage and expiry

The stability of the product is 3 years (respecting the storage conditions)

Keep the product in his package and in a cool place.

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Lukadent GmbH