



# nofire Induction wax heater

## Product description

The LUKADENT Induction wax heater is a device to apply high temperatures to waxing up instruments instantaneously in a safe convenient manner, rather than heating the waxing up instrument by open flame or with a Bunsen burner, alcohol lamps or gas burners.

- High temperatures can directly be applied to the waxing up instrument
- It is safer than instruments using open fire
- High ecologically efficiency due to the waxing up instrument sensor
- Ergonomically designed for convenient use
- High hygienic standards due to the replaceable caps
- Warning by error code, when waxing up instrument is inserted, too long, to protect electrical circuit

## Instruction for use

- 1) Plug in power cord
- 2) Turn on power-switch
- 3) Plug the waxing up instrument into the heating area
- 4) Your waxing up instrument is now heating. When heating the control LED will switch from green to red.

Power	230 V
Efficiency	130 W
Dimensions	92 x 270 x 95 mm
Weight	550 gr
REF	3819100

