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## 2.

### 3. Merkmale

- Pressure indicator to control the pressure.
- A powerful 200W motor
- Temperature adjustable between 20 - 120° C
- Digital temperature control display

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Any commissioning and handling of the device requires precise knowledge and observance of these operating instructions. The device is only intended for the use described.

#### ACHTUNG:

- Each device leaves the factory after final inspection and a test run. However, if the device is damaged or modified due to transport or other circumstances, it must not be put into operation. Check the device for damage at regular intervals. Damaged parts must be replaced immediately.
- The regulations concerning possibly required inspections (e.g. according to professional associations or electrical installations) are the sole responsibility of the purchaser/user of the product.
- This device has been designed for use in accordance with the technical description and is therefore only to be used for the intended purpose. Any other use is not permitted for safety reasons - in this case the liability of the manufacturer expires.
- The device may only be used by persons who have been instructed in its handling and who have been expressly authorized to operate it.
- The unit may only be set up in a clean room that can be heated and ventilated, if necessary under a fume hood.
- The installation surface should be level, straight and stable enough for use.
- No objects may be placed on the unit.
- To avoid any risk of injury, wear appropriate protective clothing and use appropriate auxiliary tools (such as gloves, safety glasses, mouth guard, safety shoes, ear protection and pliers).
- If the expected work results are not met, do not continue to work with the device. The owner is liable for any consequential damage caused by continued use.
- In order to prevent water, fire and explosion damage, it is essential to shut off all power sources at the end of work or during interruptions to work and to secure them against unauthorized operation.
- Always switch off the main switch and disconnect the mains plug before carrying out maintenance, cleaning and repair work and before opening the unit.
- Repairs may only be carried out by qualified and certified personnel.
- If there is any suspicion of liquid penetration, stop operation and contact the service department.
- 

#### **THE DEVICE REQUIRES A SEPARATE POWER CONNECTION!**

Make sure that the device is operated with the correct mains voltage. Connect the device directly to its own power outlet. **DO NOT** use an extension cord and/or a multiple socket outlet that is also used by other electrical equipment.

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Never open a pressurized device. Pay attention to the pressure display. Danger of splashing water, steam or adjusted parts.

Be careful when handling hot water, there is a danger of scalding. Danger of dry running and damage.

Filling of aggressive and harmful materials is not permitted.

Operate the device with a maximum pressure of 3.5 bar.

At a temperature above 100°C, the steam pressure prevents the lid from opening. Please wait until the temperature drops significantly below 100°C, switch off the heating if necessary. The pressure reduction must be completely finished. (Check pressure display or valve)

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Risk of injury from very hot water or the hot workpiece. When polymerizing with temperatures >100°C - risk of scalding by water vapor. - see point 6.2

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Liability for the function of the device is always transferred to the owner or operator if the device is improperly maintained, repaired or modified by persons who do not belong to an authorized specialist company, or if it is handled in a manner that does not correspond to its intended use. The device must be maintained and operated in accordance with these operating instructions. Lukadent GmbH is not liable for damage resulting from non-observance of the above instruction

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Hersteller: Lukadent GmbH