

Melting point: 90°C - 92°C
Pouring temperature: 45°C
Curing time: 45 min

LUKAGel Dubli instructions

Color: Red

Art.nr.: EM1216

Packaging: bucket, 6 kg.

Description: reversible duplicating gel based on agar-agar for the production of full and partial dentures and cast models and in combination with phosphate-bonded investments and ethyl silicate investments.

Use of the product: removable partial dentures, duplicating.

GMDN nr.: 16675

Handling practices and warnings: wash hands after using the product.

Eye protection: safety glasses with side-shields.

Skin protection: suitable protective clothing recommended.

Hygienic measures: do not eat, drink or smoke at the workplace.

Handle in accordance with good industrial hygiene and safety practices.

If there is the possibility of skin/eye contact the indicated hand/eye/body protection should be used. Wash face and/or hands before break and end of work.

After contact with skin, wash immediately with plenty of water.

Do not swallow product.

Contra-indication: if persons are known to be allergic to any of the components, the material should not be applied.

Storage practices and warnings:

Form: solid at room temperature.

Keep containers tightly closed and room well ventilated.

Conditions to avoid: temperatures > 50°C.

Solubility in water: good.

Incompatible materials: oxidation agents.