VESTIGE GENGIVEST / VESTIGE GENGIVEST SEPARATOR

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SOLO PER USO DENTALE

VESTIGE GENGIVEST / VESTIGE GENGIVEST SEPARATOR

LABORATORY ADDITION SILICONE FOR THE CREATION OF REMOVABLE GINGIVAL MASK ON MODELS

DESCRIPTION / INDICATIONS

Vestige gengivest: Medium consistency laboratory Vinylpolysiloxane for the reproduction of the alveolar mucosa. It is specific for works on models with implants. Being easy to trim, it allows a fast and accurate finishing touch. It can be scanned. Color: pink. Also available in the more elastic version: **Vestige gengivest soft**

Vestige gengivest separator: liquid separator to isolate Vestige gengivest

WARNINGS

Before using Vestige gengivest, it is recommended to isolate the contact surfaces with Vestige gengivest separator, in order to prevent adhesion with the materials of the same nature (example: addition silicone templates), or chemically different (condensation silicones or polyeters). Avoid contact with latex gloves.

DIRECTION FOR USE – VESTIGE GENGIVEST SEPARATOR

Vigorously shake the vial before use. Lay out Vestige gengivest separator with the brush on the contact surfaces (dental impression or silicone masks), which must be clean and dry. Wait until Vestige gengivest separator is completely dry, facilitating this operation with an air jet.

PRECAUTIONS

Do not ingest, avoid contact with eyes

DIRECTIONS FOR USE – VESTIGE GENGIVEST

Insert the cartridge into the manual dispenser. The notch "V" on the basis of the cartridge should be facing down. Lock down the appropriate device of the dispenser. Remove the cap by rotating ¼ turn counterclockwise. By acting on the lever of the dispenser, dispense a small amount of material until the two components come out evenly. Delete the extruded material and mount the mixing tip in the reverse order to the removal of the cap. Proceed according to the chosen technique (direct or indirect).

CONSERVATION

Store in a dry place at a temperature of 5-27°C (41-80°F). Protect from heat. The materials are guaranteed until the expiration date when stored in their properly closed containers. The production lot and the expiration date are indicated on the packaging.

TECHNICAL DATA	VESTIGE GENGIVEST	VESTIGE GENGIVEST SOFT
Mixing ratio	1 :1	1 :1
Mixing time (23°C/73°F)	Automixing	Automixing
Working time (23°C/73°F)	2'	2'
Setting time (23°C/73°F)	6'	6'
Hardness - Shore A	72 ± 2	40 ± 2
Linear dimensional change	< 0,20%	< 0,20%
Recovery from deformation	>99,8 %	>99,8 %

Higher temperature reduce the times, lower temperatures increase them

VESTIGE CLEAR FLD/ VESTIGE CLEAR THX

LABORATORY TRANSPARENT ADDITION SILICONE FOR MOULDING AND INJECTING LIGHT CURING COMPOSITES

DESCRIPTION / INDICATIONS

Medium consistency laboratory Vinylpolysiloxane for counter mould in composite techniques with a transparent flask. Particularly suitable for extensive work.

Vestige clear FLD: fluid version / **Vestige clear THX:** thixotropic version

WARNINGS

Avoid contact with latex gloves.

PRECAUTIONS

Do not swallow, avoid contact with eyes

DIRECTIONS FOR USE

Insert the cartridge into the manual dispenser. The notch "V" on the basis of the cartridge should be facing down. Lock down the appropriate device of the dispenser. Remove the cap by rotating ¼ turn counterclockwise. By acting on the lever of the dispenser, dispense a small amount of material until the two components come out evenly. Delete the extruded material and mount the mixing tip in the reverse order to the removal of the cap.

CONSERVATION

Store in a dry place at a temperature of 5-27°C (41-80°F). Protect from heat. The materials are guaranteed until the expiration date when stored in their properly closed containers. The production lot and the expiration date are indicated on the packaging.

TECHNICAL DATA	VESTIGE CLEAR FLD / VESTIGE CLEAR THX
Mixing ratio	1:1
Mixing time (23°C/73°F)	Automixing
Working time (23°C/73°F)	1'
Setting time (23°C/73°F)	12'
Hardness - Shore A	74 ± 2
Linear dimensional change	< 0,20%
Recovery from deformation	>99,8%

Higher temperature reduce the times, lower temperatures increase them

Limitation of the responsibility. The indications related to the products TRAYART, both in oral form and written, are founded on the knowledge and experience acquired in dental field. The professional users are kept however to follow the indications for the use and to verify the fitness of the product according to the type of application. The professional user is assumed all the responsibilities for the consequential risks from a non correct use of the products TRAYART. A possible defectiveness of the products limits the responsibility of the manufacturer to the value of the products used by the professional.

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